

*Live the Longer  
Life You Love®*

**NEW!**


## **5850 MOBILITY BED RAIL**

*The Bed Rail With Legs & A Swing Out Arm*





EASILY SIT UP IN BED

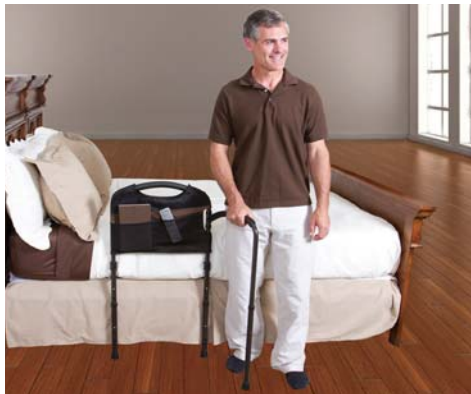

SWING-OUT ARM IMPROVES WALKING STABILITY



ORGANIZER KEEPS ITEMS CLOSE BY

## Features & Benefits

- **Swing Out Mobility Arm** – Pivots outward to provide improved walking stability.
- **Extendable Legs** – Provides added stability when standing up.
- **Ergonomic Cushion Handle** – Allows for easy transfer in and out of bed.
- **4-Pocket Organizer** – Provides storage space for handy items.
- **Anti-slip grips** – Secures rail in-between mattress and bed frame.
- **Height Adjustment** – Accommodates any home or hospital bed.
- **Easy Installation** – Installs in seconds with no tools required.

## Specifications

Height adjustment from floor to top of handle: 21" - 44"  
 Height adjustment from mattress base to top of handle: 17.5" - 22.5"  
 Rail width: 19"  
 Height range from the floor to horizontal support structure under mattress: 13.5" - 21.5"  
 Depth of horizontal support structure under mattress: 25"  
 Mobility Arm: swings out 19" away from bed  
 Weight capacity: 300 lbs  
 Weight of product: 10.5 lbs  
 Made of: Steel frame with durable powder coat finish  
 Case of 4 Dimensions: 28" L x 20.25 W x 5.75" H; 32 lbs

## How Do We Compare

Mobility Bed Rail

Other Bed Rails

Swing Out Mobility Arm



Tool-Free Assembly



Extendable Legs for Added Stability



4-Pocket Organizer Pouch Included



Provides Assistance In and Out of Bed



Durable Powder Coat Finish



## Easy Assembly (No Additional Tools Required)



Adjust base tubes to accommodate left or right side of bed



Attach base tubes to main handle



Insert base tubes between mattress and box spring



Adjust height of handle and legs as desired